

Plastic/Orthopedic



DePuy Mitek 1.3mm
Micro Anchor



● **DePuy Mitek**

a Johnson & Johnson company

IT'S ABOUT RESULTS



The DePuy Mitek 1.3mm

Superior Fixation for Precise Techniques

Hand reconstructive operations are delicate procedures. Success depends on strong tissue reattachments. The DePuy Mitek 1.3mm Micro Anchor responds to all conditions.

The 1.3mm Micro Anchor locks nonabsorbable, 4/0 white ETHIBOND® Excel surgical suture into predrilled bone sites, firmly anchoring soft tissues to bone. It fastens tightly into subcortical cancellous bone. The anchor's tiny size and simplified instrumentation provide operative flexibility and placement into previously inaccessible sites.

These qualities make the DePuy Mitek 1.3mm Micro Anchor effective in many intricate reconstructive surgeries, such as profundus tendon reattachment and collateral ligament reconstruction.

Small

Micro Anchor's miniature size facilitates placement of multiple anchors for greater intraoperative and postsurgical strength.

Versatile

Micro Anchor's design enables placement in previously inaccessible sites for greater surgical versatility.

Secure

Micro Anchor's fixation is secure, with twice the USP strength requirement for 4/0 suture.



Drill the Hole

- Using the drill, prepare the anchor's insertion site.



Place the Anchor

- Place the inserter sleeve on the perimeter of the predrilled hole.
- Establish the correct axial alignment of the inserter to the hole.



Insert the Anchor

- Holding the plastic handle of the inserter, push the thumb slide on the handle toward the drill hole until thumb slide stops, causing the anchor to implant.
- Withdraw the inserter to deploy the suture.



Set the Anchor

- Apply nominal tension (approximately 1 lb) on the lengths of suture to set the anchor in the hole.
- Use free needle to place suture in soft tissue.

The Procedure

Inserting the DePuy Mitek 1.3mm Micro Anchor

Specialized instruments facilitate placement of the DePuy Mitek 1.3mm Micro Anchor; the drill and the inserter. With these, positioning is simple and fixation is secure.

m MicroAnchor

*DePuy Mitek
Revolutionary Designs.
Multispecialty
Applications.
The Gold Standard.*

Worldwide, DePuy Mitek Anchors set the gold standard for soft tissue reattachment. DePuy Mitek's unique proprietary suture anchor product lines enhance a wide range of surgical techniques, speeding operational success. That's why DePuy Mitek is the first name in suture anchors.



Actual size:
1.3 mm

● **DePuy Mitek**

a Johnson & Johnson company

IT'S ABOUT RESULTS



Ordering Information

			
Cat. No.	Description	Needle Geometry	Quantity
212811	DePuy Mitek 1.3 mm Micro QUICKANCHOR™† - Preloaded 4/0 ETHIBOND® w/ Drill Bit (C-1)	point 	1/box
212801	DePuy Mitek 1.3 mm Micro QUICKANCHOR™ - Preloaded 4/0 ETHIBOND w/ Drill Bit (P-3)	point 	1/box
212831	DePuy Mitek 1.3 mm Micro QUICKANCHOR™ - Preloaded 3/0 ETHIBOND w/ Drill Bit (V-4)	point 	1/box
† Only available preloaded to 4/0 white ETHIBOND Excel suture and potassembled with a single-use inserter and drill. QUICKANCHOR is a registered trademark of DePuy Mitek, Inc.			

 Physician's authorization

 Date

For more information on the DePuy Mitek 1.3mm Micro Anchor,
 call your DePuy Mitek representative or **1-800-382-4682**.



IT'S ABOUT RESULTS



DePuy Mitek, Inc., 325 Paramount Drive, Raynham, MA 02767, USA

© DePuy Mitek, Inc., 1997. All rights reserved. Printed in USA. P/N 900199 Rev. C 1/01